




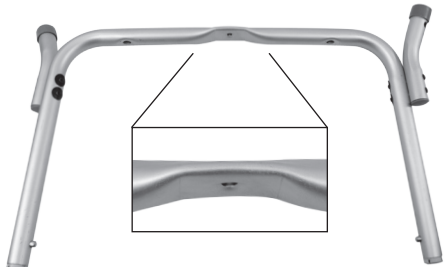

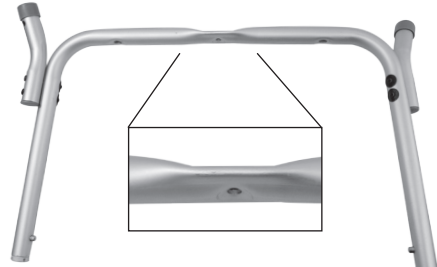


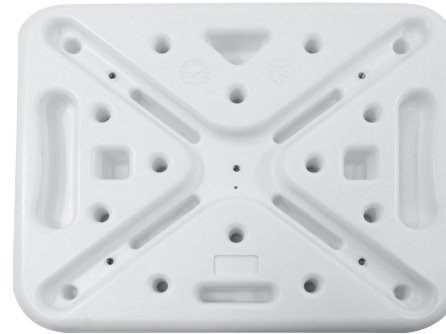



B3011 Shower Bench with Backrest

Included in box:
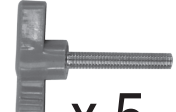
<div>A</div> <div></div> <div>Frame Tubing (A)</div>	<div>B</div> <div></div> <div>Frame Tubing (B)</div>	<div>C</div> <div></div> <div>Seat Panel</div>	
<div>D</div> <div></div> <div>Backrest Panel</div>	<div>E</div> <div></div> <div>x 2</div> <div>Armrest</div>	<div>F</div> <div></div> <div>x 2</div> <div>Backrest Support Tubing</div>	
<div>G</div> <div></div> <div>x 4</div> <div>Adjustable Leg</div>	<div>H</div> <div></div> <div>x 5</div> <div>Short Wing Knob Screws</div>	<div>I</div> <div></div> <div>x 4</div> <div>Long Wing Knob Screws</div>	<div>J</div> <div></div> <div>x 9</div> <div>Wave Washers</div>

ATTENTION:

- Please verify that all components are included as shown above, before proceeding with assembly.
- **DO NOT USE AS A VANITY STOOL**

USER WEIGHT LIMIT: 300lbs.


WARNING:

- Before using, be sure that all gold locking buttons fully protrude thru holes of each Adjustable Leg (G).
- Make sure all Adjustable Legs are snapped in the same hole position (G).
- Periodically retighten all wing knob screws on the bottom of the seat.


California Proposition 65

⚠ WARNING: Cancer & Reproductive Harm
www.P65Warnings.ca.gov


- 1** Place the Seat Panel (C) upside down on a flat surface with the triangle indent facing upwards. This triangle indicates the front of the Seat Panel (C).

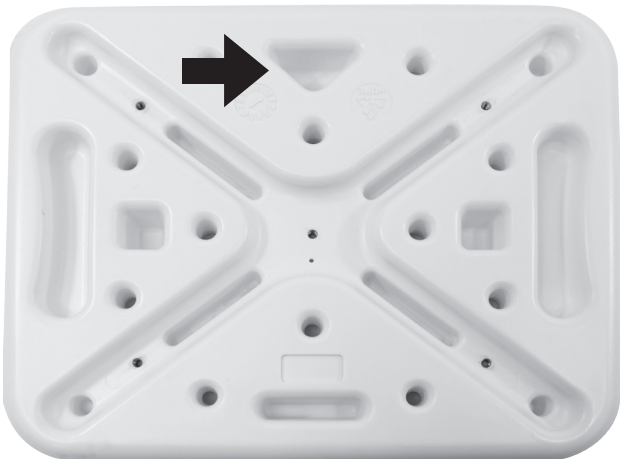


- 2** Identify the difference between the upper and lower Frame Tubing pieces (A & B). Both of the pieces have stickers to identify them. One Frame Tubing piece is labeled "A" and the other is labeled "B".

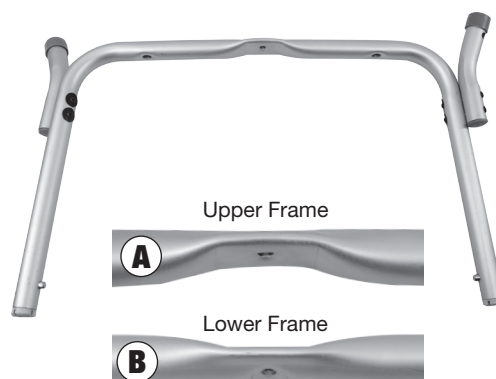


- 3** Place upper Frame Tubing piece "A" in the indent labeled "A", with the legs facing upwards.

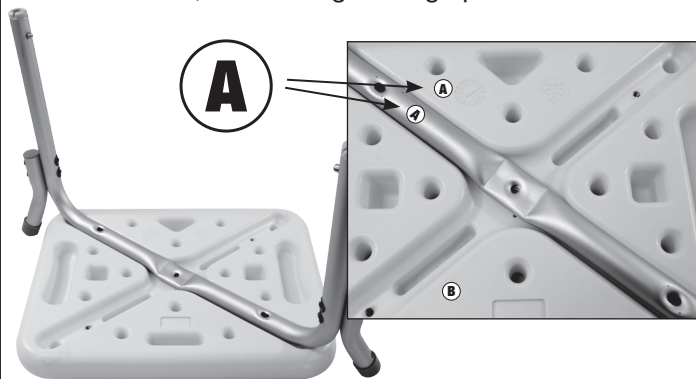


- 4** Place lower Frame Tubing piece "B" in the indent labeled "B" with the legs facing upwards crossing over upper Frame Tubing piece "A".

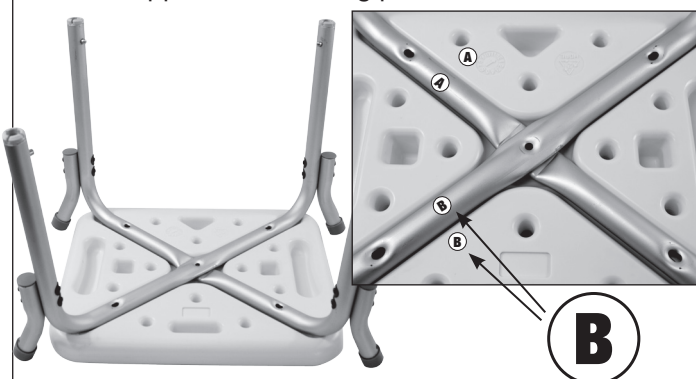

- 5** Take one of the Short Wing Knob Screws (H) and Wave Washers (J) and place the Wave Washer (J), with arc facing down, onto the Wing Knob Screw (H). Screw the Short Wing Knob Screw (H) into the hole where the two Frame Tubing pieces (A & B) overlap. Hand tighten the Wing Knob Screw (H) while making sure the Wave Washer (J) aligns with the tubing.

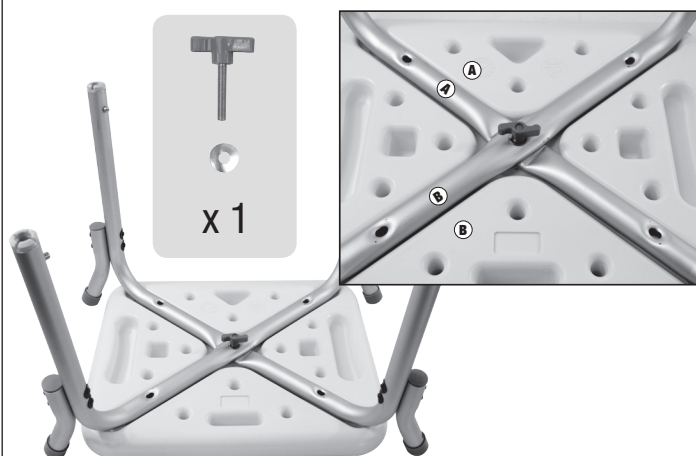


- 6** Identify the two different sides of the Backrest Support Tubing (F). One side will have holes that are farther apart while the other has holes that are closer together.

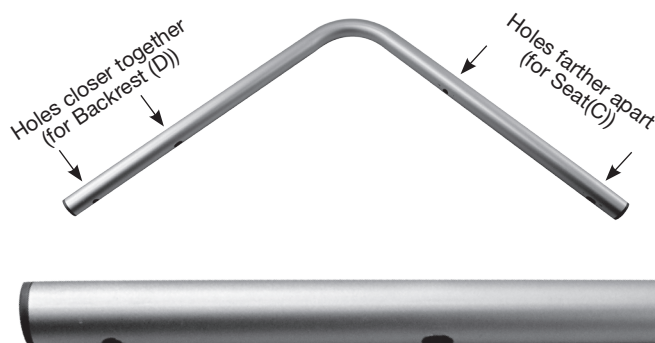

Place the side, with the holes separated farther, over the preexisting holes of the upper Frame Tubing (A & B). Make sure the "L" portion of the Backrest Support Tubing (F) is facing the back of the Seat Panel (C) and downwards during assembly.

- 7** Take two Long Wing Knob Screws (I) and two Wave Washers (J) and place the Wave Washers (J) on the Wing Knob Screws as in Step 5. Screw the Long Wing Knob Screws (I) into the holes where the Back Support Tubing (F) overlaps with the Frame Tubing pieces (A & B). Hand tighten both Wing Knob Screws. Repeat this with the remaining Back Support Tubing (F), Long Wing Knob Screws (I) and Wave Washers (J).

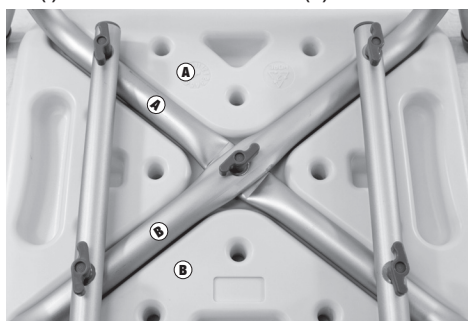

- 8** Identify the brass buttons on the four posts of the Frame Tubing (A & B). Take one of the Adjustable Legs (G) and while pressing in the button, slide the Adjustable Leg (G) over the post until the brass button protrudes. Adjust height accordingly and repeat with remaining Adjustable Legs (G).



- 9** Flip chair over and verify all Adjustable Legs (G) are touching the floor. Make sure the chair is level and stable.



- 10** Identify the top of the Backrest Panel (D). The top portion has a handle molded into it. Take remaining Short Wing Knob Screws (H) and Wave Washers (J) and place Wave Washers (J) on the Short Wing Knob Screws (H) as in Step 5. Line up the holes of the Backrest Panel (D) to the holes on the Backrest Support Tubing (F) with top of Backrest Panel (D) up. Take the Short Wing Knob Screws (H) and Wave Washers (J) and fasten into the hole. Hand tighten.



- 11** Take the Armrests (E) and identify the brass buttons. Press the brass buttons in while sliding the Armrest (E) into the frame silencers. Make sure the buttons protrude from the holes in the Frame Tubing (A & B). Repeat with second Armrest (E).



- 12** Now that the chair is completely assembled, ensure all of the Wing Knob Screws (H & I) are tightened.

WARNING:

- Before using, be sure that all gold locking buttons fully protrude thru holes of each Adjustable Leg (G).
- Make sure all Adjustable Legs (G) are snapped in the same hole position.
- Periodically retighten all wing knob screws on the bottom of the seat.



Essential Medical Supply, Inc. Limited Warranty

Essential Medical Supply, Inc. ("Essential") warrants its products free from defects in materials and workmanship ("Warranty") for a Limited Lifetime from the date of purchase ("Warranty Period"). The Warranty extends only to the original purchaser of Essential products. If within the Warranty Period there is a breach of Warranty as to an Essential product, such product shall be repaired or replaced at Essential's sole discretion. This Warranty does not include any labor or shipping costs incurred in replacement, part installation, or repair of any Essential product. **ESSENTIAL'S SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO REPAIR AND/OR REPLACEMENT OF THE DEFECTIVE ESSENTIAL PRODUCT.**

For Warranty service, please contact the dealer from whom you purchased your Essential product. In the event you do not receive satisfactory Warranty service, please write Essential directly at:

Essential Medical Supply, Inc.
Customer Service Department
6420 Hazeltine National Drive
Orlando, FL 32822-5121

Please provide the dealer's name, address, and date of purchase, and indicate the nature of the problem and the Essential product serial number.

Upon receipt of written notification, Essential will issue a return authorization. The defective unit or parts must be returned to Essential for Warranty inspection using the serial number as identification within thirty (30) days of the date of return authorization. **DO NOT** return products to Essential without Essential's prior written consent. Cash on delivery shipments will be refused and shipping charges must be prepaid.

Limitations and Exclusions: The Warranty shall not apply to products if 1) the Essential serial number has been removed or defaced; 2) the product has been damaged by negligence, accident, improper operation, maintenance or storage; and/or 3) the product has been modified without Essential's express written consent or repaired by an unauthorized third-party ("Warranty Disqualifying Events"). The Warranty Disqualifying Events include but are not limited to the addition of unauthorized parts or attachments or damage caused by circumstances beyond Essential's control, the occurrence of which shall be determined exclusively by Essential. The Warranty also shall not apply to problems or defects arising from normal wear and tear on Essential products and parts, including but not limited to rubber tips and hand grips, and failure to adhere to user instructions and safety precautions.

THE WARRANTY STATED HEREIN IS THE SOLE AND EXCLUSIVE WARRANTY OFFERED BY ESSENTIAL. THERE ARE NO OTHER WARRANTIES RESPECTING ESSENTIAL'S PRODUCTS, EITHER EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO ANY WARRANTY OF DESIGN, MERCHANTABILITY, OR FITNESS FOR A PARTICULAR PURPOSE, EVEN IF ESSENTIAL HAS BEEN INFORMED OF SUCH PURPOSE. NO AGENT OF ESSENTIAL IS AUTHORIZED TO ALTER OR EXCEED ESSENTIAL'S WARRANTY OBLIGATIONS AS SET FORTH HEREIN. ESSENTIAL SHALL NOT BE RESPONSIBLE FOR ANY INDIRECT, INCIDENTAL, SPECIAL, OR CONSEQUENTIAL DAMAGES ARISING OUT OF USE OF ESSENTIAL PRODUCTS.

Product Serial No. _____